

Other Implant Relative

IMPLANT POSITIONING KIT SD-IPO

Possible to place implant inducing without guide stent.

Predictable of the site and size of crown engaged finally in a process of implant surgery.

Separated drill and guide to engage each needed.



Positioning drill

- Engaged with guide to drill on the site for dental implantation.



Positioning guide

- Guide tools to induce implant to be placed on the proper site.



* When connected



Guide pin

- In surgery of multiple implants, it is used to make the surgical site or path for Implant fixtures accurately and to check the size or shape of crown.



How to use

- Select the Positioning guide that matches dimension of the missing tooth and drill on the site connecting with Positioning drill to make an orifice.
- Replace Positioning drill with the same size Guide pin.
- Select the Positioning guide that matches dimension of the second missing tooth.

Process of surgical operation from the next step is same as ① to ②.

